

Colonovideoscope

# CF-EZ1500DL/I

Full Focus and High Magnification for Observational Excellence





## Full Focus and High Magnification for Observational Excellence

### EDOF Technology

Our unique Extended Depth of Field (EDOF) technology combines images captured in near view with those taken in far view to generate an image with a wider depth of field that gives you greater clarity and richer detail throughout the image area, assuring superior observation with continuous wide focusing.


### Full Focus

Thanks to EDOF technology, the CF-EZ1500DL/I can approach as close as 3 mm to the subject in normal focus mode, providing a clear, sharp view that may reduce the need to adjust the focus in routine use.

### High Magnification with the Ease of Dual Focus


The CF-EZ1500DL/I offers high magnification\* when the Near Focus mode is engaged via a simple scope switch, supporting better observation of details. The combination of high magnification and wide depth of field allows for clearer, more detailed imaging throughout the field of view, even from a tangential direction and regardless of whether the subject is moving or not. In combination with our proven NBI technology, the CF-EZ1500DL/I's advanced optics is designed to assist accurate, high-confidence optical diagnosis.

\*Maximum magnification of 75x (with OEV262H) and 90x (with OEV321UH)

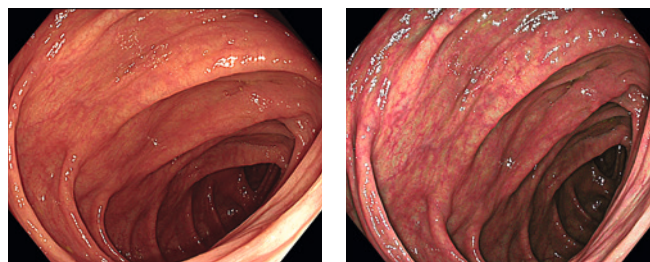

White Light Normal Focus

TXI

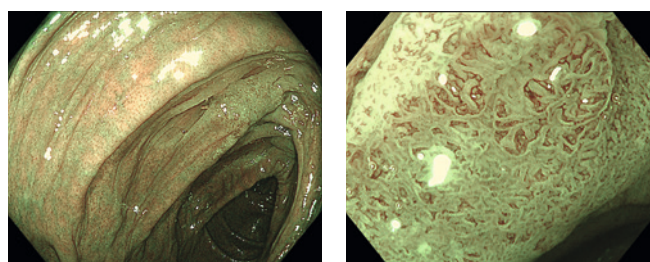

NBI Normal Focus

NBI Near Focus

### Other Features

- Compatible with TXI, RDI and NBI when connected to CV-1500
- ErgoGrip
- RIT (Responsive Insertion Technology)
- ScopeGuide-ready
- 170° Wide viewing angle
- WaterJet
- Waterproof one-touch connector

Specifications			
<b>Optical System</b>	Field of view	Normal focus mode: 170° Near focus mode: 160°	<b>Bending Section</b> Angulation range Up 180° / Down 180° / Right 160° / Left 160°
	Direction of view	Forward viewing	
	Depth of field	Normal focus mode: 3 - 100 mm Near focus mode: 1.5 - 5.5 mm	
<b>Insertion Section</b>	Distal end outer diameter	ø 13.2 mm	<b>Total Length</b> L: 2005 mm I: 1655 mm
	Distal end enlarged		
<b>Instrument Channel</b>	Channel inner diameter	ø 3.7 mm	<b>Compatible System</b> Video system center OLYMPUS CV-1500
	Minimum visible distance*1	4 mm (Normal focus mode)	
	Direction from which EndoTherapy accessories enter and exit the endoscopic image		
<b>Auxiliary Water Channel</b>	Direction from which water jet appears in the endoscopic image		

\*1 Distance from the distal end of the endoscope.  
\*2 Standard when CV-1500 (high air pressure) is used.



COLONOVIDEOSCOPE OLYMPUS CF-EZ1500DL/I

Specifications, design, and accessories are subject to change without any notice or obligation on the part of the manufacturer.